



MILLIKEN HEALTHCARE PRODUCTS, LLC

BEST CLINICAL PRACTICES

Milliken Healthcare Products, LLC TRITEC™, ULTRA, and ASSIST dressings have the versatility to be used across a wide range of wound types and are effective from low to high exuding wounds. Every TRITEC, ULTRA and ASSIST dressing utilizes Active Fluid Management® Technology, a three part design that combines two layers of high performance fabric using a micro-knit process. This creates a powerful moisture transfer mechanism that pulls away and traps excess exudate while leaving an optimal amount of moisture at the wound bed.

TRITEC	Size	Code	HCPCS	Packaging
	4"x5"	3000020040	A6207	10 ea/bx 10 bx/cs
	4"x48"	3000020051	A6207	10 ea/cs
TRITEC Silver	Size	Code	HCPCS	Packaging
	1"x24"	3000004574	A6207	5 ea/bx 10 bx/cs
	4"x5"	3000005137	A6207	10 ea/bx 10 bx/cs
	4"x5" Pre-Cut	3000112331		10 ea/bx 10 bx/cs
	6"x6"	3000004573	A6207	10 ea/bx 10 bx/cs
	4"x48"	3000004575	A6208	10 ea/cs
	8"x16"	3000005211		50 ea/cs

ULTRA	Size	Code	HCPCS	Packaging
	4"x5"	3000006840	A6210	10 ea/bx 10 bx/cs
ULTRA Silver	Size	Code	HCPCS	Packaging
	4"x5"	3000051018	A6210	10 ea/bx 10 bx/cs
ASSIST Silver	Size	Code	HCPCS	Packaging
	6"x6"	3000084768		10 ea/bx 10 bx/cs
	4"x48"	3000051175		10 ea/cs
	8"x20"	3000051176		10 ea/bx 3 bx/cs
	24"x36" Silver Absorbent Pad	3000006964		20 ea/bx

WOUND TYPE	Shallow/Deep Pressure Ulcers	Diabetic Ulcer	Venous Ulcer	Arterial Ulcer	Partial Thickness Burn	Skin Graft/LSE	Surgical Incision
WOUND DESCRIPTION	Superficial, often appearing as an abrasion, blister, or shallow skin loss	Ulcer of the foot caused by pressure or stress typically accompanied by neuropathy or loss of sensation	Ulceration of the lower leg, caused by venous insufficiency, typically accompanies by edema and heavy exudate	A lower extremity ulcer caused by impairment or arterial blood flow, often resulting in tissue ischemia	Burn that extends through the epidermis and into the dermis	Healthy skin or tissue-engineered skin used in burn and chronic wound treatment	A wound or cut as a result of a surgical procedure
WOUND VISUAL							
PRIMARY DRESSING	TRITEC Silver 4x5, 6x6 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 1x24 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 6x6, 4x48 TRITEC 4x5, 4x48 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 4x48 TRITEC 4x5, 4x48 ULTRA Silver 4x5 ULTRA 4x5	ASSIST 4x48, 6x6, 8x20 TRITEC Silver 4x5 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	ASSIST 4x48, 6x6, 8x20 TRITEC Silver 4x5 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 1x24 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5
SECONDARY DRESSING	ULTRA 4x5, gauze or equivalent absorptive dressing Wound veil/filler as needed	ULTRA 4x5, gauze or equivalent absorptive dressing	ULTRA 4x5, gauze or equivalent absorptive dressing May be used under compression	ULTRA 4x5, gauze or equivalent absorptive dressing	ULTRA 4x5, gauze or equivalent absorptive dressing Wound veil suggested (Wound veil not needed with ASSIST)	ULTRA 4x5, gauze or equivalent absorptive dressing Wound veil suggested (Wound veil not needed with ASSIST)	ULTRA 4x5, gauze or equivalent absorptive dressing

The information provided herein is not to be construed as the practice of medicine or substituted for the independent medical judgment of a patient's treating physician. This information, including but not limited to suggestions for product usage, product selection and suggested use is based on independent generalization and does not consider the unique characteristics of an individual's wound. Each patient's physician shall remain solely responsible for assessing the severity of patient wounds, determining the appropriate treatment and managing treatment of the wound. Please see product insert for complete list of instructions.